




TravelScoot® Smart Luggage Kit

Instruction

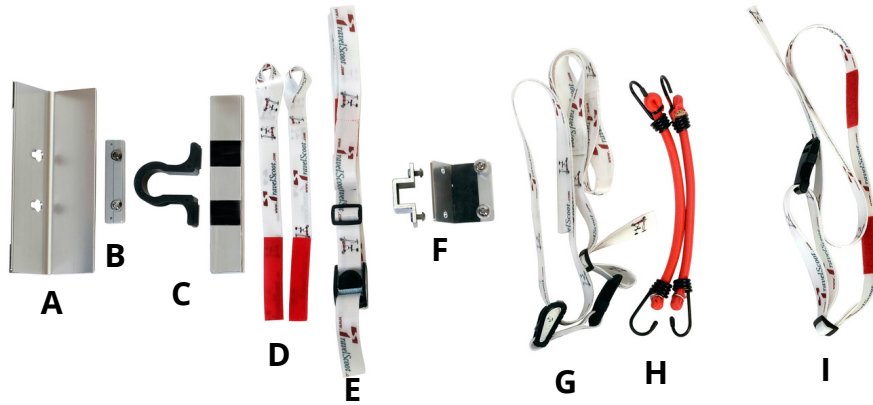

Note: For Escape and Shopping Cruiser, 1.) and 2.) are obsolete, begin with 3.)

1.) Remove this part if installed (see arrow)



2.) Insert **Adapter Rail (B)** into the U-shaped basket rail with the smooth side pointing aft. Insert the two small bolts into the holes (these serve as retaining pins).

3.) Slightly loosen the round-head bolts. Align the keyhole slots of the **support platform (A)** with the bolt heads, and slide the support platform over the bolts. Slide the support platform downward into place and tighten the bolts.



4.) Snap the omega-shaped plastic **spacer (C)** onto the steering column. Attach the aluminum tube strip with the attached Velcro.



Travel

5.) Loop the two **short straps (D)** around the suitcase handle. The straps can remain attached to the suitcase.



6.) Attach the **long strap (E)** to the rear surface of the support platform by joining the Velcro patches.

Please note: To prevent the strap and spacer from getting lost during airline transit, remove these items and place them in your carry-on luggage prior to boarding.





7.) Raise the TravelScoot handlebars to the highest setting and lock the lever clamp. Place luggage on the **support platform (A)**, loop the **short straps (D)**, around the steering column, and pull tight against the **spacer (C)**.

The brake cables and electronic control wires must be free, and the handlebar free to move. If they are obstructed, that means that the suitcase exceeds IATA size limits.


Pull the strap up over the suitcase and tighten the buckle.

8.) Bolt the **wheel-case hook attachment (F)** to the lowest point of the backrest.



9.) Hang the wheel-case onto the hook.

Once seated, you can place another piece of luggage below the seat.

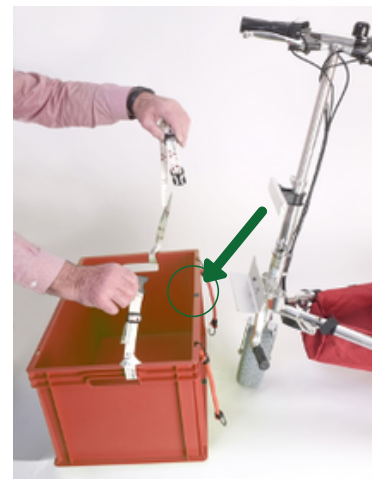

Attention! Do not drive backwards with the case attached!

Shopping

10.) Loop the **straps (G)** through the crate handles, and mark the center of crate's upper edge. →



11.) Place the crate on the **support platform (A)** and align the center mark with the steering column. Place the straps around the **omega-shaped spacer (C)**, connect the clips and pull to tighten. Attach the two **bungees (H)** between the crate and the cane holders. The bungees reduce the possibility of lateral tilting.

Caution: A crate this size (as discussed in step 11.) can hold over 40 lbs. Due to the TravelScoot's light weight of only 35 lbs, weight distribution is very important. A fully loaded crate

could cause the TravelScoot to tip forward when the rider dismounts. Therefore, we recommend carrying additional, loaded containers at the rear of the scooter as ballast.

Please note: When removing a loaded front-mount crate, please have a second person help support the crate before opening the clasp and unhooking the bungees (H).

12.) **Additional belt (I)** to use as needed.

