



# TravelScoot® Hook-On

## Instructions for initial set-up and installation



1.) Apply both adhesive loop-Velcro strips to the vertical part (legs, below the bend) of the seat yoke with the cut ends facing inward.

2.) Mark the center of the battery / batteries using e.g. a marker or piece of tape.




3.) Remove the seat from the yoke and set it upside down. Slide the 3cm long (approx.. 1 1/8 inch) seat spacer over the seat post. Now slide the flat metal hook plate over the seat post with the hooks pointing towards the bottom of the seat, followed by the 2cm (3/4 inch) spacer. These parts may be secured from falling off during seat removal using tape or adhesive.

4.) Our folding baskets feature a clasp on the upper edge for securing when folded. This clasp interferes with the Hook-on accessory, so please install the basket with the clasp facing away from the hook, or remove the clasp altogether.



# Hook-on assembly instructions for two folding baskets and an additional battery

1.) Place the first battery in its bracket with the wire pigtail towards the upper left. Place the two H-brackets onto the battery within the two markings. The longer hooks must point forward.

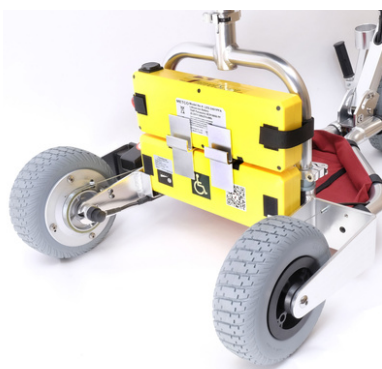

2.) Insert the second battery and align with the first, with the wire pigtail towards the LOWER left. This way the batteries do not need to swap positions to connect to the scooter. Secure the upper battery to the yoke with the hook-Velcro strips.

3.) Open the folding baskets and press down the bottom surfaces. The split section (see arrow) is the support surface for the lower flat hooks.



4.) Install the seat in the yoke. Install the first basket by engaging the upper and lower flat hooks simultaneously. Repeat with the second basket.